


DuraMatrix®

Restore with confidence

DuraMatrix-Onlay® PLUS

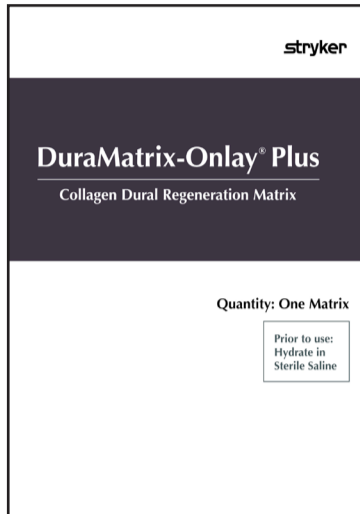


Characteristics


- Application*:** Onlay application. Smooth side up, sponge side down (toward the brain)
- Material:** Purified type I collagen derived from bovine Achilles tendon
- Hydration:** Prehydrate in sterile saline
- Thickness:** Approx 3.5mm thick
- Resorption:** Resorbs in approx. 8 weeks^{1,2}

Part numbers and sizes

Part number	Size	Units per
DMOP11	1 in x 1 in (2.5 cm x 2.5 cm)	1
DMOP13	1 in x 3 in (2.5 cm x 7.5 cm)	1
DMOP22	2 in x 2 in (5 cm x 5 cm)	1
DMOP33	3 in x 3 in (7.5 cm x 7.5 cm)	1
DMOP45	4 in x 5 in (10 cm x 12.5 cm)	1
DMOP57	5 in x 7 in (12.5 cm x 17.5 cm)	1

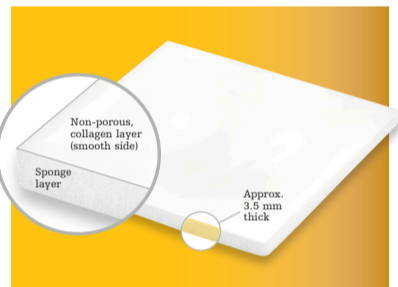

DuraMatrix-Onlay® Suturable

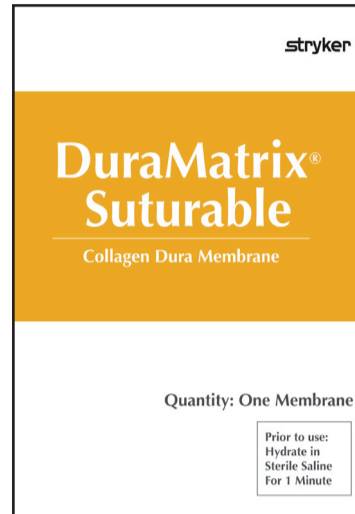

Characteristics


- Application*:** Suturable application.
- Material:** Intact bovine dermis collagen derived from bovine Achilles tendon
- Hydration:** Prehydrate 1 minute in sterile saline
- Thickness:** Approx 1.0mm thick
- Resorption:** Resorbs in approx. 38-40 weeks^{1,2}

Part numbers and sizes

Part number	Size	Units per
DMS11	1 in x 1 in (2.5 cm x 2.5 cm)	1
DMS13	1 in x 3 in (2.5 cm x 7.5 cm)	1
DMS22	2 in x 2 in (5 cm x 5 cm)	1
DMS33	3 in x 3 in (7.5 cm x 7.5 cm)	1
DMS45	4 in x 5 in (10 cm x 12.5 cm)	1



DuraMatrix-Onlay®

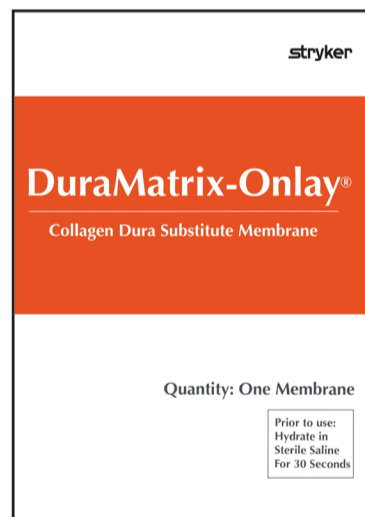

Characteristics


- Application*:** Onlay application.
- Material:** Purified type I collagen derived from bovine Achilles tendon
- Hydration:** Prehydrate 30 seconds in sterile saline
- Thickness:** Approx 0.6mm thick
- Resorption:** Resorbs in approx. 6-9 months^{1,2}

Part numbers and sizes

Part number	Size	Units per
CDSL11	1 in x 1 in (2.5 cm x 2.5 cm)	1
CDSL13	1 in x 3 in (2.5 cm x 7.5 cm)	1
CDSL22	2 in x 2 in (5 cm x 5 cm)	1
CDSL33	3 in x 3 in (7.5 cm x 7.5 cm)	1
CDSL45	4 in x 5 in (10 cm x 12.5 cm)	1
CDSL57	5 in x 7 in (12.5 cm x 17.5 cm)	1



DuraMatrix®

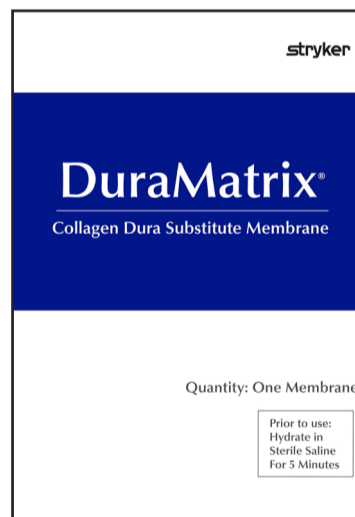

Characteristics

- Application*:** Onlay and suturable application.
- Material:** Purified type I collagen derived from bovine Achilles tendon
- Hydration:** Prehydrate 5 minutes in sterile saline
- Thickness:** Approx 0.3mm thick
- Resorption:** Resorbs in approx. 6-9 months^{1,2}

Part numbers and sizes

Part number	Size	Units per
CDSM11	1 in x 1 in (2.5 cm x 2.5 cm)	1
CDSM13	1 in x 3 in (2.5 cm x 7.5 cm)	1
CDSM22	2 in x 2 in (5 cm x 5 cm)	1
CDSM33	3 in x 3 in (7.5 cm x 7.5 cm)	1
CDSM45	4 in x 5 in (10 cm x 12.5 cm)	1



References: 1. Rabbit duraplasty study: Data on file at Collagen Matrix, Inc. 2. The results of pre-clinical and in vitro studies may not be indicative of human clinical outcomes. *refer to respective IFUs for detailed application information.

Craniomaxillofacial

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it in surgery. The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders. DuraMatrix-Onlay Plus and Dura Matrix Suturable are trademarks of Collagen Matrix, Inc. DuraMatrix-Onlay and Dura Matrix are registered trademarks of Collagen Matrix, Inc. DuraMatrix-Onlay Plus, Dura Matrix Suturable, DuraMatrix-Onlay and DuraMatrix are manufactured by Collagen Matrix, Inc., Oakland, New Jersey USA. DuraGen Plus is a registered trademark of Integra LifeSciences Corporation. Durepair is a registered trademark of TEI BBiosciences, Inc. The products depicted are CE marked in accordance with applicable EU Regulations and Directives. This material is not intended for distribution outside the EU and EFTA.

CMF-PO-9-EN Rev. None_12708

SDL 04/2020

SMACC Number

Copyright © 2020 Stryker

stryker.com

Collagen Matrix, Inc.
15 Thornton Road Oakland,
NJ 07436 USA

CE
2797

EC REP

MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

Distributed by:
Stryker Leibinger GmbH & Co. KG
Bötzingen Straße 41
79111 Freiburg
Germany