

Triathlon® Trial Cutting Guide (TCG)

Surgical overview

Delivering workflow efficiency in Revision TKA:


1. Create tibial platform

Augment as necessary to establish tibial platform. Insert trial augments and trial tibial baseplate.



2. Establish joint line and extension gap

Reference the medial epicondyle to set joint line. Fill extension gap with appropriate trial insert. Pin through slot.

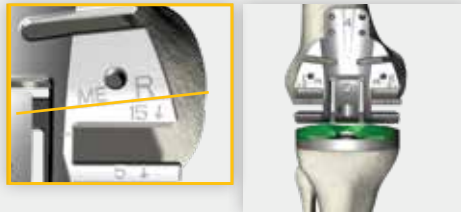

3. Evaluate flexion gap

Dynamically offset flexion gap to restore posterior condylar offset.



4. Establish femoral rotation

Rotate component to establish appropriate rotation based on EPI line and patella tracking.




5. Make augment and box cuts


After trialing, visualize and perform clean up cuts or augment cuts through the TCG. Perform box cut.




TCG tray layouts

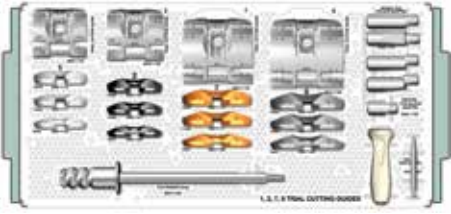


Size 1,2,7,8 TCG Instrument & Trials Upper Tray

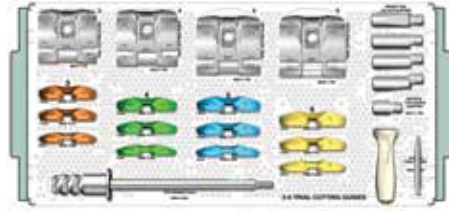

Size 3-6 TCG Instrument & Trials Upper Tray

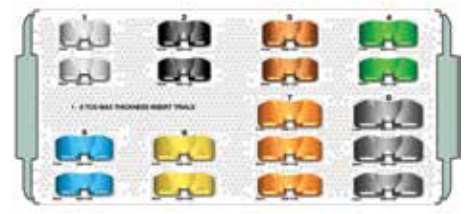

Size 1-8 TCG Max Thickness Insert Trials

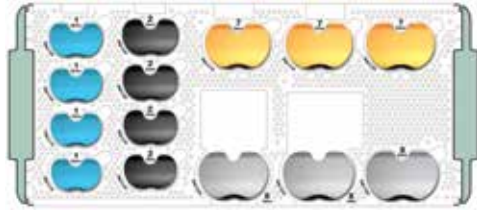

Size 1,2,7,8 TCG Instrument & Trials Lower Tray



Size 3-6 TCG Instrument & Trials Lower Tray

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker's product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any of Stryker's products. The products depicted are CE marked according to the Medical Device Regulation 2017/745 or the Medical Device Directive 93/42/EEC. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your sales representative if you have questions about the availability of products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Triathlon. All other trademarks are trademarks of their respective owners or holders.