

Antibacterial Body Cleansing Washcloths

with 2% Chlorhexidine Gluconate

REF 9602-ANZ



Features and benefits

Antibacterial Body Cleansing Washcloths

2% Chlorhexidine Gluconate

- Helps reduce bacteria¹
- Provides rapid bactericidal action¹
- Rinse-free
- Non-alcohol and non-alkaline base

Specifications

Product name	Antibacterial Body Cleansing Washcloths with 2% Chlorhexidine Gluconate
Product code	9602-ANZ
Description	Six (6) polyester disposable washcloths
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Dimensions of each wipe	19 cm x 19 cm
Amount of packages per case	40

Safety directions / Warnings / First aid:

Keep out of reach of children.

Read safety directions.

- For external use only.
- Keep out of eyes, ears and mouth. May cause serious injury if chlorhexidine is permitted to enter and remain. If contact should occur, flush with water and consult a doctor.
- Stop use and consult a doctor if irritation, sensitisation, or allergic reaction occurs. In rare cases severe allergic reaction may occur.
- Do not use on skin that is compromised beyond the superficial layers.
- Do not use immediately prior to invasive techniques such as venepuncture or lumbar puncture.
- Use with care in newborn babies, especially those born prematurely. Antibacterial Body Cleansing Washcloths with 2% Chlorhexidine Gluconate may cause chemical skin burns.

Not made with natural rubber latex.

Do not flush.

Single use only.

References:

1. Mutters, N., Gunther, E., Kaiser, S., Fries, T. and Frank, U. 2015. Is your antiseptic effective against clinical multidrug-resistant microorganisms? A chlorhexidine digluconate formulation demonstrates efficacy even in lower concentrations. *Antimicrobial Resistance and Infection Control*, 4(S1).

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it. The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area. Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders. For more information please contact your local Stryker sales representative.

SG_2826_SAG
Copyright © 2021 Stryker

Packaging information

Contents:

(6) Disposable Washcloths

Indications of Use:

- Helps reduce bacteria¹
- Provides rapid bactericidal action¹


Instructions:

Use one washcloth for each area of the body. Do not rinse. **Do not swallow.** If desired, warm packages in the microwave (1000W) **no longer than 30 seconds.** **Caution:** Test temperature of washcloths prior to use. Wait if excessively hot.

1. Neck, Chest
2. Arms
3. Groin
4. Legs
5. Back
6. Buttocks

Ingredients:

Water (aqua)
Propylene glycol
Glycerin
Chlorhexidine gluconate 2%
Aloe barbadensis (Aloe vera)
Gluconolactone
Dimethicone
Nonoxynol 9
Polysorbate 20
Fragrance (parfum)

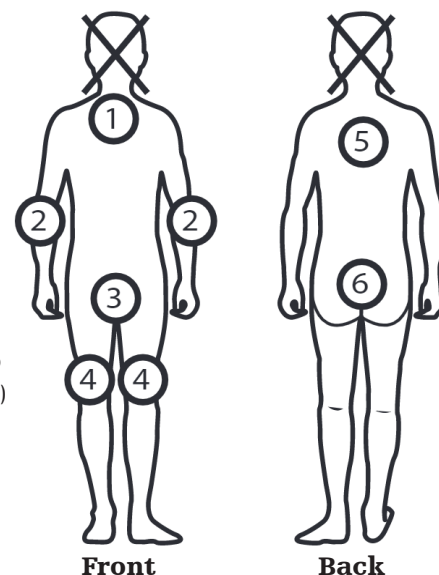

Front

Back

Stryker Australia Pty Ltd
8 Herbert Street
St Leonards NSW 2065
61 2 9467 1000
www.stryker.com.au

Stryker New Zealand Limited
PO Box 17-136, Greenlane
Auckland 1546, New Zealand
64 9 573 1890
www.stryker.com